

dentally

Dentally Vision v175

RCI Full Release notes

To benefit from updates marked with '**AA update required**' you **MUST** update your Acquisition Agent. Keep in mind that your acquisition agent is always backwards compatible and you will still benefit from all other updates listed below.

Need help updating or want to learn more? Visit our [Help Center](#) for details on updating your agent.

New features

Compare up to six images side-by-side


Clinicians can now compare 2, 4, or 6 X-ray images in a single view, making it easier to review changes over time.

What's changed?

- Switch between comparison layouts without losing already selected images.
- Clear colour and number indicators will show which image slot is selected
- Linked zooming across images makes side-by-side insights faster and more consistent
- Full functionality is now available in full-screen compare view!

Why it matters?

Faster clinical comparison with better visual clarity and less rework when reviewing images.

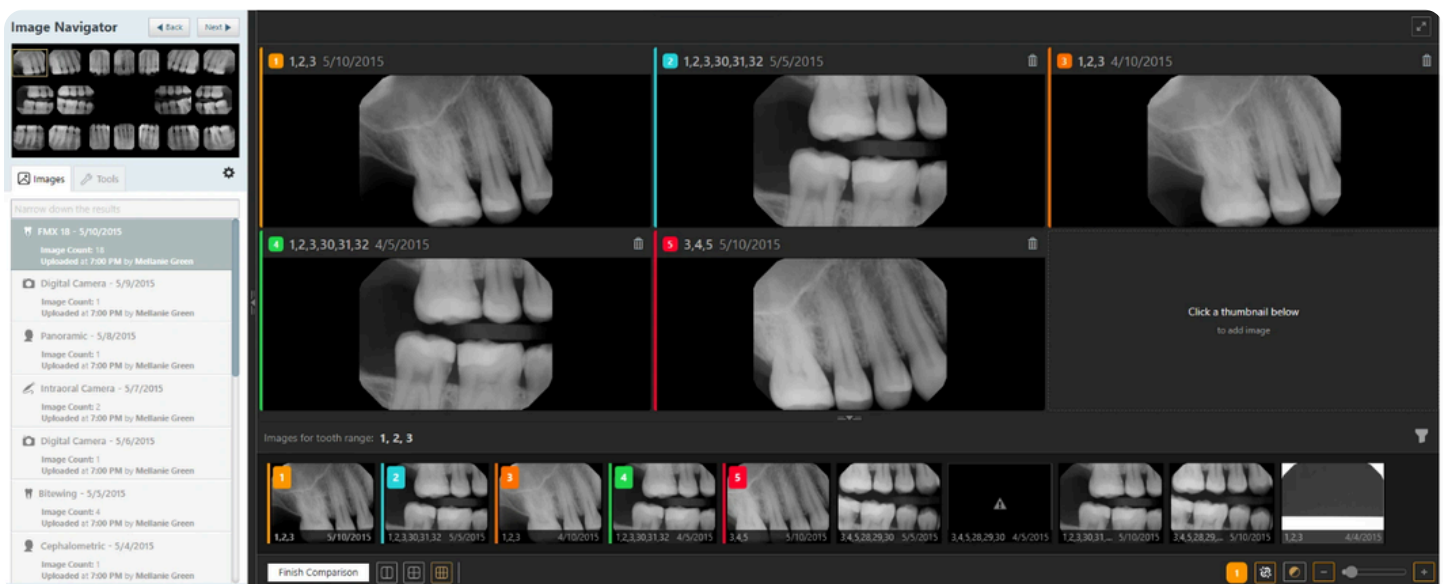

Improvements

Image header readability

We have made some small yet meaningful improvements to how image details are displayed.

What changed?

- The **date text size now matches the tooth number**, improving visual consistency
- Applies across imaging views where both are shown

Why it matters?

Easier scanning of image details, especially during fast-paced clinical reviews.

Procedure menu enhancements

The procedure menu now makes better use of available screen space allowing you to see more valuable information without needing to scroll.

What's changed?

- The menu automatically expands to use unused vertical space
- More procedures are visible at once, with scrolling only when needed

Why it matters?

Less scrolling, faster access, and a smoother workflow during imaging sessions.

Bug fixes

Image & exam handling

- Fixed issues opening exams that contained deleted images
- Improved handling of images with measurements and transformations
- Resolved issues where the first image in a PSP exam could appear twice
- Enhancement filters now default correctly (off) for Vistascan and PSPIX devices.

Image compare & adjustments

- Brightness and contrast adjustments now work consistently across all compared images
- Filtering in image comparison now behaves correctly and shows the expected images
- AI buttons no longer appear when AI features are disabled
- Image compare action buttons now display correctly when comparing images

Layout & display fixes

- Images now resize and re-center correctly when side panels are opened or collapsed
- Tool panels (AI, density, calibration) display correctly at smaller screen sizes